


Heritage II Bariatric

Quick Reference Guide

This Quick Reference Guide (QRG) has been produced to assist users with only the functional aspects of the system. Please refer to the product User Manual for full instructions for use.

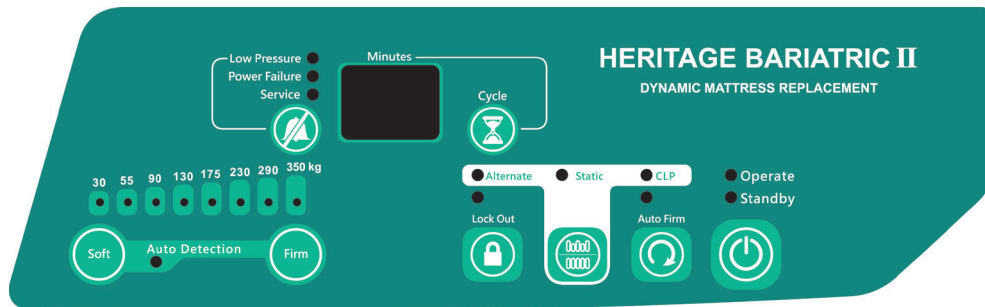

Control Panel Lock Out

To unlock panel:

- Press & hold for 3 seconds to unlock control panel.


To lock panel:

- To lock out the control panel press the **Lock Out** button and all functions will be locked.
- If the control panel is not used for a period of 30 seconds it will automatically lock out and inhibit the use of the functions.



Comfort Control

The **Soft** and **Firm** buttons allow adjustment of pressures within a safe pre-set range.

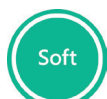

- When pressing the **Firm** button, the output pressure will increase to provide a higher pressure output and thus increased support.
- When pressing the **Soft** button, the output pressure will be decreased to provide a lower pressure output and thus increased comfort.



Auto Firm Mode

- Pressing the **Auto Firm** button quickly inflates all cells to a single pressure setting.
- The system will automatically return to **Alternating** mode after 20 minutes.



Function Mode Button

- Pressing the **Mode** button allows you to cycle between the different modes available such as **Alternating**, **Static** and **Constant Low Pressure**.



Alternating Mode

- **Alternating** mode is the default mode for the system. Within this mode the mattress will inflate and deflate in an alternating 1-in-2 cell cycle.
- The alternating cycle will continue at the selected cycle time until another mode is selected.



Static Mode

- Pressing the **Mode** button until the **Static** indicator illuminates puts the system into **Static** mode.
- After 30 minutes the system will automatically revert back to **Alternating** mode.



Constant Low Pressure (CLP) Mode

- Pressing the **Mode** button until the **CLP** indicator illuminates puts the system into **Constant Low Pressure** mode.



Alternating Cycle Time Selection

- Choice of 10/15/20/25/30 minutes.
- To select press **Cycle** button until required cycle time is displayed in LCD screen.

Auto-Detection of Pressure Settings

- The Heritage II Bariatric features **Auto-Detection** of cell pressures to provide appropriate care for each patient.
- Utilised through '**Auto**' activation or '**Manual**' activation.

Auto-Detection

- Every 2 hours system will automatically check for changes in patient weight/position, during this time the **Auto-Detection** indicator will flash.
- Once the system has finished its detection it will set the internal cell pressures accordingly and the indicator light will remain ON.



Manual Auto-Detection

- Simultaneously pressing the **Soft** and **Firm** buttons on the control panel will manually activate **Auto-Detection**.
- Once activated the indicator light will flash and the system will start detecting the patients weight to set internal pressures appropriately.

