

Height Adjustable Toilet Frame with Injection Moulded Seat

UTOF0003



USER MANUAL



WARNING: Please read this Manual carefully before using the **Height adjustable toilet frame with injected moulded seat** as it contains important information regarding the safe use of the product.

Contents

1.	Introduction	4
2.	Description of product	4
3.	Fitting instructions	4
4.	Cleaning	5
5.	Care and maintenance	5
6.	Technical Specification	6
7.	Warranty	7
8.	Disposal advice	7
9.	Life expectancy.....	7
10.	Contact Us	8

1. Introduction

These instructions aid the user and/or carer about the safe and efficient use of this product, including assembly, cleaning and maintenance, and operation of the product.

2. Description of product

This Toilet Frame supports clients to give them confidence as they sit & rise, offering support and lessening of strain. It has an adjustable height feature to cater for varying patient requirements. The easy to clip on / clip off seat makes it easy to fit & remove and the partial cutaway provides easy access for personal cleansing. This product is suitable for intensive cleansing.

3. Fitting Instructions

- a) Carefully unpack and check the product to ensure it is complete.
- b) Before use, check for any signs of damage e.g., cracked weld, flaking paint, or parts bent. If any damage is seen, please contact your Supplier.
- c) Remove the pin clips from the legs and safely put to one side.
- d) Select the desired height of the frame by gently sliding out each extension leg from the frame.
- e) When the desired height is obtained, refit the pin clips into the appropriate hole ensuring that the pin clips go through each leg and its extension leg and that all legs are set to the same height.
- f) Push the moulded seat onto the frame making sure that it is positioned as centrally as possible.
- g) The frame is now ready for use.



NOTE:

- The legs are extended telescopically.



CAUTION:

- Do not attempt to use if the product shows any sign of damage.

4. Cleaning

Use mild detergent with warm water to wipe down the framework and seat on at least a weekly basis or more frequently if required.



CAUTION:

- ☐ Abrasive cleaners and/ or abrasive cleaning pads will damage the frame
- ☐ If the product is cleaned using a power/ jet wash or industrial washing machines care must be taken since high-pressure water can be penetrative. The frame should not be placed too close to high-pressure water jets.
- ☐ After cleaning, ensure that any residual water is removed from the framework as this can lead to permanent rusting of the frame. (Leave frames open in a warm environment for several hours to ensure that all internal moisture evaporates.)
- We cannot offer advice on industrial washing machines with regards to specific machine settings as these will be dependent on the manufacturer's instructions

5. Care and Maintenance

The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking of paint. Any signs of damage should be noted and reported to your supplier.

6. Technical specification

Product code	UTOF0003
Toilet seat height	438 - 585mm
Width between arms	468mm
Toilet seat depth	419mm
Centre of seat to top of arms	226mm
Footprint	530 - 585mm x 529mm
Aperture	260 x 230mm
Maximum user weight	190kg/ 30st

7. Warranty

The Height adjustable toilet frame with injected moulded seat is warranted for 12 months from the date of purchase against manufacturing or design defects.

The warranty is subject to the following conditions:

- It does not apply to wear and tear, inappropriate or incorrect use.
- Defective set up by the user or third parties. Negligent or careless treatment.
- Use of unsuitable or unapproved spares or unauthorised alteration.
- The warranty does not extend to consequential costs resulting from fault clearance, loss of earnings or expenses

This in no way affects your statutory rights.

8. Disposal advice

This product must be thoroughly cleaned/sterilised prior to disposal. The product and /or its components are of standard non-hazardous materials therefore can be disposed of without any potential environmental hazards providing that local recycling arrangements are sought and adhered to.

9. Life expectancy

We estimate a life expectancy of 2 years for this product, provided that:

- It is used in strict accordance with the intended use as set out in this document.
- All service and maintenance requirements are met.

The estimated life expectancy can be exceeded if the product is carefully used and properly maintained.

The life expectancy can also be considerably reduced by extreme or incorrect usage.

The fact that we estimate a life expectancy for this product does not constitute an additional warranty.

10. Contact Us

Ultimate Healthcare customer service is available to answer questions from 8.30am to 5.00pm, Monday to Friday for business and upgrades. Our sales team is available for pricing and order processing and status queries. Our Technical Support team is available to answer your queries related to troubleshooting, please contact the following:

Ultimate Healthcare Limited

Nutsey Lane
Calmore Industrial Estate
Totton,
Southampton,
SO40 3XJ

Tel: 0333 321 8996

Email: Enquiries@ultimatehealthcare.co.uk

Please Note: Information in this user manual is subject to change without notice and does not represent commitment on the part of Ultimate Healthcare Limited. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording for any purpose without the written permission of Ultimate Healthcare.

Ultimate Healthcare Limited.
Calmore Industrial Estate
Totton, Southampton
SO40 3XJ
United Kingdom



Doc no.
UTOF0003.IFU-3

Issue 3.0
May 2025

www.ultimatehealthcare.co.uk