

# Perching Stools

UHOS1002-2, UHOS1003-2, UHOS1004-2



---

**USER MANUAL**

---



**WARNING:** Please read this Manual carefully before using the **Perching Stools** as it contains important information regarding the safe use of the product.

## Contents

1.	Introduction .....	4
2.	Description of product .....	4
3.	Instructions for use .....	4
4.	Cleaning .....	5
5.	Care and maintenance .....	5
6.	Technical Specification .....	6
7.	Warranty .....	7
8.	Disposal advice .....	7
9.	Life expectancy.....	8
10.	Contact Us .....	9

## 1. Introduction

These instructions aid the user and/or carer about the safe and efficient use of this product, including assembly, cleaning and maintenance, and operation of the product.

## 2. Description of product

The Ultimate Healthcare perching stool offers comfortable and reliable support. The features of a back, arms and sloped seat give added support and its splayed legs offer increased stability.

## 3. Instructions for use

To adjust the height, remove the 'E' clip from the extension leg and extend all legs evenly to the desired height.

Once a desired height has been reached, reinstall the 'E' clip. Ensure the 'E' clip has passed through the extension leg and that it fits well around the outside leg.



### NOTE:

- The perching stool is designed to slant towards the front.
- When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat.

**CAUTION:**

- Do not exceed the last punched hole of the extension leg.
- Do not adjust any of the legs to make the seat level. This will cause excessive stress to the legs compromising user safety and possibly leading to permanent damage of the perching stool.
- Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.
- Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.
- Do not use the perching stool outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.

## 4. Cleaning

Prior to cleaning the perching stool remove all pin clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

Clean the upholstery using a non-abrasive soft cloth with a mild detergent.

**CAUTION:**

- Abrasive cleaners and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair.

## 5. Care and Maintenance

This product is maintenance free so long as the cleaning and safety instructions shown above are adhered to.

Check the product regularly to ensure all components are present and not damaged and to ensure that it is securely assembled and suitably tightened.

## 6. Technical specification

	Perching stool with vinyl seat & arms	Perching stool with vinyl seat & arms & back	Perching stool with vinyl seat & arms & padded back
Product code	UHOS1002-2	UHOS1003-2	UHOS1004-2
Width (mm)	560 - 635		
Depth	415 - 460		
Seat width	325		
Seat depth	265		
Seat height	545 - 685		
Width between arms	505		
Height of arms from seat	175		
Max user weight	165kg   26st		

## **7. Warranty**

The Perching Stools are warranted for 12 months from the date of purchase against manufacturing or design defects.

### **The warranty is subject to the following conditions:**

- It does not apply to wear and tear, inappropriate or incorrect use.
- Defective set up by the user or third parties. Negligent or careless treatment.
- Use of unsuitable or unapproved spares or unauthorised alteration.
- The warranty does not extend to consequential costs resulting from fault clearance, loss of earnings or expenses

**This in no way affects your statutory rights.**

## **8. Disposal advice**

This product must be thoroughly cleaned/sterilised prior to disposal. The product and /or its components are of standard non-hazardous materials therefore can be disposed of without any potential environmental hazards providing that local recycling arrangements are sought and adhered to.

## 9. Life expectancy

We estimate a life expectancy of 2 years for this product, provided that:

- It is used in strict accordance with the intended use as set out in this document.
- All service and maintenance requirements are met.

The estimated life expectancy can be exceeded if the product is carefully used and properly maintained.

The life expectancy can also be considerably reduced by extreme or incorrect usage.

The fact that we estimate a life expectancy for this product does not constitute an additional warranty.



## 10. Contact Us

Ultimate Healthcare customer service is available to answer questions from 8.30am to 5.00pm, Monday to Friday for business and upgrades. Our sales team is available for pricing and order processing and status queries. Our Technical Support team is available to answer your queries related to troubleshooting, please contact the following:

### **Ultimate Healthcare Limited**

Nutsey Lane  
Calmore Industrial Estate  
Totton,  
Southampton,  
SO40 3XJ

Tel: 0333 321 8996

Email: [Enquiries@ultimatehealthcare.co.uk](mailto:Enquiries@ultimatehealthcare.co.uk)

**Please Note:** Information in this user manual is subject to change without notice and does not represent commitment on the part of Ultimate Healthcare Limited. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording for any purpose without the written permission of Ultimate Healthcare.

---

Page left intentionally blank

---

---

Page left intentionally blank

---

Ultimate Healthcare Limited.  
Calmore Industrial Estate  
Totton, Southampton  
SO40 3XJ  
United Kingdom



Doc no.  
UHOS1001-2.3.4.IFU-2

Issue 2.0  
May 2025

[www.ultimatehealthcare.co.uk](http://www.ultimatehealthcare.co.uk)