

# Toilet Frame and Clip-on Seat with Chute

UTOF0002-2



---

**USER MANUAL**

---



**WARNING:** Please read this Manual carefully before using the **Toilet Frame and clip-on seat with chute** as it contains important information regarding the safe use of the product.

**Contents**

<b>1.</b>	<b>Introduction .....</b>	<b>4</b>
<b>2.</b>	<b>Description of product .....</b>	<b>4</b>
<b>3.</b>	<b>Fitting instructions .....</b>	<b>4</b>
<b>4.</b>	<b>Cleaning .....</b>	<b>5</b>
<b>5.</b>	<b>Care and maintenance .....</b>	<b>5</b>
<b>6.</b>	<b>Technical Specification .....</b>	<b>6</b>
<b>7.</b>	<b>Warranty .....</b>	<b>7</b>
<b>8.</b>	<b>Disposal advice .....</b>	<b>7</b>
<b>9.</b>	<b>Life expectancy.....</b>	<b>7</b>
<b>10.</b>	<b>Contact Us .....</b>	<b>8</b>

## 1. Introduction

These instructions aid the user and/or carer about the safe and efficient use of this product, including assembly, cleaning and maintenance, and operation of the product.

## 2. Description of product

This product has been designed to allow the seat & chute to be easily removed for easy cleaning of all parts, helping to minimise infection control issues. The frame is coated with a tough anti-scratch paint finish for improved longevity. It is also provided with a splash guard (Chute) and is designed to fit over standard toilet suites. The frame is height adjustable and therefore suits a range of different users. The frame has integral arms to aid safe transfer and non-slip ferrules are supplied as standard.

## 3. Adjusting the height

Remove the e-clips on each of the legs and either lift or lower the frame to the desired height. Once the desired height has been achieved, replace the tube clips in the nearest hole in both tubes locking the frame in position.



### **NOTE:**

- The legs are extended telescopically.

## 4. Cleaning

Use a non-abrasive mild detergent with warm water and a soft cloth to clean the frame and seat. Rinse with clean warm water & a soft dry cloth.



### **NOTE:**

- This product is suitable for thermal decontamination in an automated washing machine.



### **CAUTION:**

- Abrasive cleaners and/ or abrasive cleaning pads will damage the frame.

## 5. Care and Maintenance

This product is maintenance free so long as the cleaning and safety instructions shown above are adhered to.

Check the product regularly to ensure all components are presently, for damage, to ensure that it is securely assembled and suitably tightened.

## 6. Technical specification

<b>Product code</b>	UTOF0002-2
<b>Maximum height</b>	820mm
<b>Minimum height</b>	690mm
<b>Width</b>	540mm
<b>Depth</b>	370mm
<b>Width between arms</b>	495mm
<b>Weight</b>	4kg
<b>Maximum user weight</b>	190kg/ 30st

## 7. Warranty

The Toilet Frame and clip-on seat with chute are warranted for 12 months from the date of purchase against manufacturing or design defects.

### **The warranty is subject to the following conditions:**

- It does not apply to wear and tear, inappropriate or incorrect use.
- Defective set up by the user or third parties. Negligent or careless treatment.
- Use of unsuitable or unapproved spares or unauthorised alteration.
- The warranty does not extend to consequential costs resulting from fault clearance, loss of earnings or expenses

**This in no way affects your statutory rights.**

## **8. Disposal advice**

This product must be thoroughly cleaned/sterilised prior to disposal.  
The product and /or its components are of standard non-hazardous materials therefore can be disposed of without any potential environmental hazards providing that local recycling arrangements are sought and adhered to.

## **9. Life expectancy**

We estimate a life expectancy of 2 years for this product, provided that:

- It is used in strict accordance with the intended use as set out in this document.
- All service and maintenance requirements are met.

The estimated life expectancy can be exceeded if the product is carefully used and properly maintained.

The life expectancy can also be considerably reduced by extreme or incorrect usage.

The fact that we estimate a life expectancy for this product does not constitute an additional warranty.

## 10. Contact Us

Ultimate Healthcare customer service is available to answer questions from 8.30am to 5.00pm, Monday to Friday for business and upgrades. Our sales team is available for pricing and order processing and status queries. Our Technical Support team is available to answer your queries related to troubleshooting, please contact the following:

### **Ultimate Healthcare Limited**

Nutsey Lane  
Calmore Industrial Estate  
Totton  
Southampton  
SO40 3XJ

Tel: 0333 321 8996

Email: [Enquiries@ultimatehealthcare.co.uk](mailto:Enquiries@ultimatehealthcare.co.uk)

**Please Note:** Information in this user manual is subject to change without notice and does not represent commitment on the part of Ultimate Healthcare Limited. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording for any purpose without the written permission of Ultimate Healthcare.

Ultimate Healthcare Limited.  
Calmore Industrial Estate  
Totton, Southampton  
SO40 3XJ  
United Kingdom



Doc no.  
UTOF0002-2.IFU-3

Issue 3.0  
May 2025

[www.ultimatehealthcare.co.uk](http://www.ultimatehealthcare.co.uk)