

# Heritage II Digital

## Dynamic Mattress Replacement



The Heritage II Digital dynamic mattress replacement system provides optimal therapy for the prevention and treatment of pressure ulcers in any care environment. Featuring an intuitive digital control panel the Heritage II Digital is easy to operate and ensures high quality care for patients up to a very high risk.



### Key zoning and therapy to address patient needs

The Heritage II Digital is constructed of deep 8" cells, in a cell-on-cell design and provides dual therapy modes - **Alternating** and **Static** for patients up to 255kg/40 stone.

Specific zones of the mattress address the key comfort and care needs of each patient. Static head cells provide support and stability, the lower leg zone provides lower residual interface pressure in this highly sensitive area, while dynamic therapy is provided in the main body area.



### High quality pressure care for high risk patients

Powered by **dual compressors** the digital power unit of the Heritage II Digital offers a host of key features that deliver high quality pressure care for patients at High to Very High Risk of pressure damage.

**Variable cycle times** offer the choice of a standard 10 minute cycle or extended periods of 15/20/25/30 minutes, which may be desirable as patients begin to rehabilitate. The system offers both **Alternating** and **Constant Low Pressure** therapy modes, while the ability to select the **Manual Auto-Detection** feature allows patient weight/position to be automatically detected and pressure set accordingly, ensuring optimum patient protection.



### Enhanced patient safety and peace of mind

The Heritage II Digital addresses key patient safety issues with an **Auto Lockout** feature which prevents unintentional activation of the control panel and an excellent up to 24 hour **Transport mode** which provides a security buffer to allow patients to remain on an inflated surface in the event of a power failure.

The **Timed Static mode** automatically returns to Alternating mode safeguarding patients against the risk of pressure damage and sophisticated audible and **visual alarms** provide real peace of mind.

Constant Low Pressure mode allows the mattress to conform closely a patient's body and redistribute body weight over a wider area; this greater patient immersion delivers optimum constant levels of pressure redistribution.

## Features

### Power Unit


- Digital control panel
- Comfort control
- Constant low pressure
- Manual Auto-Detection pressure setting
- Dual 7 litres per minute air flow
- Variable cycle times (10/15/20/25/30 mins)
- Automatic control panel lockout
- Timed static mode
- Auto-Firm
- Transport mode - up to 24 hours
- Audible and visual alarms
- Annual service indicator
- Adjustable hanging brackets

### Mattress

- Cell-on-cell construction
- Static head cells
- Lower leg zone
- CPR Valve
- Strong cell retention knitted bands
- Maximum patient weight 255kg / 40 stone

### Cover

- Soft, waterproof vapour permeable PU cover
- 360 degree zipper
- Welded seams and enclosed zip
- Integral covered air pipe
- Cable management system

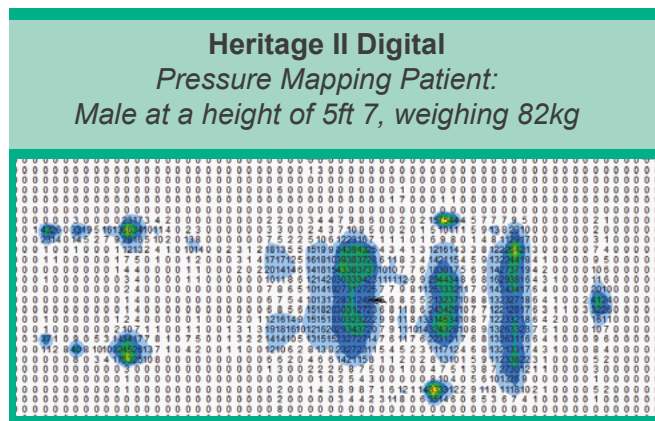

## Technical specification

|                               |                                   |
|-------------------------------|-----------------------------------|
| <b>Product code:</b>          | UPRA3678D-2                       |
| <b>Risk category:</b>         | Up to very high risk              |
| <b>Max user weight:</b>       | 255 kg / 40 stone                 |
| <b>Mattress dimensions:</b>   | 2000mm x 890mm x 200mm            |
| <b>Power unit dimensions:</b> | 360mm x 130mm x 230mm             |
| <b>Product weight:</b>        | 7.4kg Mattress / 4.6kg Power Unit |
| <b>Cell material:</b>         | Nylon PU / TPU (lower leg zone)   |
| <b>Cover material:</b>        | Polyester PU                      |

|                        |  |
|------------------------|--|
| <b>Cycle time:</b>     | 10 / 15 / 20 / 25 / 30 mins (2-cell cycle) |
| <b>Pressure range:</b> | 25-60mmHg                                  |
| <b>Supply voltage:</b> | 230V, 50 Hz                                |
| <b>Fuse rating:</b>    | T1A  |
| <b>Current rating:</b> | 0.12 A                                     |
| <b>Classification:</b> | Class II, Type BF                          |
| <b>Compliance:</b>     | EN 60601-1, EN 60601-1-2                   |
| <b>Warranty:</b>       | 2 years                                    |

### Ultimate Healthcare

Calmore Industrial Estate, Nutsey Lane, Totton, Hampshire, SO40 3XJ

tel: 0333 321 8996, email: enquiries@ultimatehealthcare.co.uk

[www.ultimatehealthcare.co.uk](http://www.ultimatehealthcare.co.uk)